

# FS-2NL

## LATEX-FREE FACE SHIELD WITH LOUPE CLEARANCE



### Cleaning Instructions:

This face shield may be used multiple times by the same person.

1. Carefully wipe the inside and then the outside of the visor using a clean cloth saturated with neutral detergent solution, rinse if needed.
2. Carefully wipe the outside of the visor using a clean cloth or wipe saturated with hospital disinfectant solution; be sure it remains wet for the required contact time. The CDC recommends a .1% chlorine solution. Carefully avoid the foam cushion and elastic strap as they may not be tolerant to disinfectants.
3. Wipe the outside of visor with clean water to remove residue.
4. Fully dry (air dry or use clean absorbent towels).

### Materials:

#### LOUPE CLEARANCE

Approximately 3"-3.5" from front of face

#### CLEAR SHIELD

Clear fog-free, glare-free polyester film, .007" thick, with or without removable protective film

#### ELASTIC HEADBAND

1" wide latex-free elastic

#### BROW FOAM

2.75" thick, soft latex-free polyurethane

**NOT MRI SAFE**

### Manufacturer:

SOUND DEVICES, LLC  
E7556 State Road 23 and 33  
Reedsburg, WI 53959  
Phone: 1-800-505-0625  
Local Phone: +1 (608) 524-0625  
Fax: +1 (608) 524-0655



Order at **store.sounddevices.com** or email **sales@sounddevices.com**.